

MVAP Medical Supplies, Inc.

1415 Lawrence Dr.

Newbury Park, CA. 91320

Cleaning MVAP ECG Cables and Leadwires

- ECG cables and leadwires should be cleaned with green prep soap, green soap tincture, or ordinary alcohol-free hand soap. After cleaning, wipe the cables and wires with a clean cloth dampened with water, then with a clean dry cloth.
- Never immerse MVAP Medical ECG cables and leadwires in any solution, and never clean with any harsh chemicals like Acetone.
- If disinfection is required, a clean cloth dampened with either water and 10% Chlorine Bleach, or 2% Glutaraldehyde Solution (such as Cidex) is recommended.