



238 Littleton Rd, Suite 202
Westford, Massachusetts 01886
399 0208 Fax

+1 978 399 0206 Main
+1 978

EzeNet® Headpiece Cleaning and Disinfecting

Always wear gloves.

General cleaning after use on intact skin:

- The EzeNet® may be cleaned by washing it in detergent.
- Be sure to keep the connector dry. It may be wiped down with alcohol.
- Rinse out material thoroughly and hang by the cable to dry, or blot with a towel.

When disinfecting is required:

- Wash first, following the above steps. Blot with a towel.
- A level one sterilization solution may be used. Disinfect the EzeNet® per the solution manufacturer's instructions. Always follow the recommended precautions.
- Do not use bleach. This will damage the EzeNet®.
- Rinse out material thoroughly and hang by the cable to dry.

When sterilization is required:

- Cold gas sterilization may be used. The EzeNet® should not be exposed to temperatures above 80F.
- We recommend disposing of the EzeNet® if contaminated with blood. Follow your facility's procedure.

Note - when used on a patient with an incurable disorder such as Creutzfeldt-Jakob disease, the EzeNet® should be disposed of.